

Headache Therapy & Specialty Appliances

| | Indications | Contraindications | What to Send | Overview |
|----------------------------------|---|---|---|--|
| NTI-tss Plus™ | <ul style="list-style-type: none"> • Migraine/tension headaches • Tooth wear from bruxing and clenching • Muscle pain associated with parafunction • Diagnostic treatment planning • TMD | <ul style="list-style-type: none"> • Advanced periodontal disease • Edge-to-edge bite • Mobile teeth | <ul style="list-style-type: none"> • Full arch VPS impressions, models or digital files • Maximum protrusive measurement • Centric occlusal bite | <ul style="list-style-type: none"> • Provides instant and continual incisal guidance in centric clench and any excursions. By eliminating canine and posterior contact, muscle contraction intensity is reduced by 70%. The standard design covers the centrals and laterals, but can be extended to cover any number of teeth. |
| NTI-tss Plus Soft | <p>Same as NTI-Plus when patient has:</p> <ul style="list-style-type: none"> • Crowned anteriors • Veneers • Slight mobility | <ul style="list-style-type: none"> • Advanced periodontal disease • Edge-to-edge bite • Severely worn centrals • Incompatible occlusal scheme • Short clinical length | <ul style="list-style-type: none"> • Full arch VPS impressions, models or digital files • Maximum protrusive measurement • Centric occlusal bite | <ul style="list-style-type: none"> • The NTI-tss Plus Soft features a hard exterior with a soft interior extended to the 2nd bicuspid. It is guaranteed to reduce chairtime and increase patient comfort while protecting teeth, muscles, and joints from parafunction's devastating effects. |
| Comfort H/S™ Bite Splint | <ul style="list-style-type: none"> • Prevents damage from parafunction • Protects teeth and restorations • Provides the ability to diagnose occlusal concerns | <ul style="list-style-type: none"> • Advanced periodontal disease • Severe bruxism • Short clinical crown • Lack of undercut for retention • Multiple edentulous areas | <ul style="list-style-type: none"> • Full arch VPS impressions, models or digital files • 2mm open bite registration | <ul style="list-style-type: none"> • Protect your restorations and increase patient compliance with the comfort and durability of Comfort H/S. Its hard exterior will withstand your patient's parafunction while the soft liner and economical price guarantee they will wear it. |
| Thermofit™/ Thermo-Guard™ | <ul style="list-style-type: none"> • Prevents damage from parafunction • Protects teeth and restorations • Provides the ability to diagnose occlusal concerns | <ul style="list-style-type: none"> • Advanced periodontal disease • Mobile teeth | <ul style="list-style-type: none"> • Full arch VPS impressions, models or digital files • 2mm open bite registration | <ul style="list-style-type: none"> • Protect your restorations and minimize chairtime with this rigid, yet flexible splint. Immerse in hot water for 10 seconds, and it drops right in. No internal adjustments needed. When it is in the mouth, it quickly returns to a rigid state. |
| Crystal Clear® | <ul style="list-style-type: none"> • Prevents damage from parafunction • Protects teeth and restorations • Provides the ability to diagnose occlusal concerns • Severe bruxism • Short clinical length | <ul style="list-style-type: none"> • Advanced periodontal disease • Mobile teeth | <ul style="list-style-type: none"> • Full arch VPS impressions, models or digital files • 2mm open bite registration | <ul style="list-style-type: none"> • The world's strongest bite guard is available in multiple designs and will not absorb stains and odors. |

